

Monoprotect® INIBSA

Dental needles

Choose a high quality dental needle that provides a precise injection.



Smooth injections to comfort your patients.

Monoprotect® INIBSA

Dental needles



Stainless steel
cannulas with
silicon-coated tip.



Triple beveled
to facilitate tissue penetration
and reduce tissue injuries.



Bevel indicator
printed on the cone that
allows a proper needle positioning.



Latex free.
Reduces the risk of allergic
reactions or skin irritations.



Sterility guarantee.
Welded point that joins
safety cap and protector.



Color-coded.
Needles individually packed
in boxes of 100 pieces.



Quick identification.
Production batch, expiry
date and size printed directly
on the needle cap.



Manufactured in EU.
Metric thread.

DESCRIPTION	CODE	SIZE	CANNULA DIAMETER (mm)	NEEDLE LENGTH (mm)
● MONOPROTECT® INIBSA 30G 12MM EXTRASHORT	5283	30G	0,3	12
● MONOPROTECT® INIBSA 30G 16MM EXTRASHORT	5284	30G	0,3	16
● MONOPROTECT® INIBSA 30G 21MM MEDIUM	5285	30G	0,3	21
● MONOPROTECT® INIBSA 30G 25MM MEDIUM	5286	30G	0,3	25
● MONOPROTECT® INIBSA 27G 36MM LONG	5282	27G	0,4	36

Instructions for use:

Positioning the needle on the syringe.

- 1) Break the guarantee seal.
- 2) Pull while twisting to remove the cap.
- 3) Screw the threaded base onto the syringe, taking care not to remove the protector.
- 4) Remove the protector with a slight twisting motion just before anaesthesia.

After use:

- 1) Carefully remove the needle from the syringe to avoid accidental puncture.
- 2) Dispose of the used needle immediately in a suitable container conform to regulations, without bending it, break it or handle it in any other way.



INIBSA DENTAL S.L.U.
Ctra. Sabadell a Granollers km 14,5
08185 Lliçà de Vall - Barcelona (Spain)
Tel. +34 93 860 95 00 Fax +34 938 437 389
email: export@inibsa.com

TECNOFAR SPA
Via al Piano 54 A, 23020
Gordona (SO) - Italy
www.tecnofar.it



www.inibsa.com